



Features & Benefits



MIP's Evacuation Sheet is designed to remain underneath the mattress, ready for rapid evacuation of patients or residents in an emergency situation. The sheet works by wrapping around the patient, cocooning them within the mattress, to form a protective sledge. Drag loops at head and foot are used to tow the patient from the building. There are two alternative securing methods available to retain the evacuation sheet under the mattress, either hook and loop straps that secure the sheet to the bed frame or elastic straps that loop around each end of the mattress. In the event of soiling the evacuation sheet can easily be removed for laundering.

Safe working load: 400kg.

In normal use, your Evacuation Sheet does not require any maintenance. Periodic inspection is recommended. Attention should be given to stitching, webbing and fabric condition as degradation is possible over an extended period.

Product has been FR tested to EN12952.

Latex free

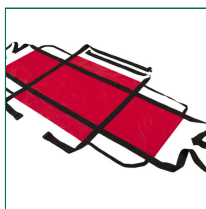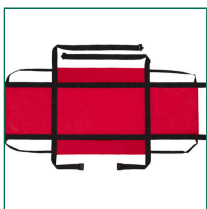

Specifications

STYLE	COLOUR/PATTERN	DESCRIPTION	SIZE
CS653-0001	Red	Evacuation Sheet	185 x 85 cm