



Connected Bed Rail Protectors

Fitting Instructions




Scan the QR code to watch the fitting video.


Ensure the bed is at a safe and comfortable working height for yourself .



1. Remove the mattress from the bed.

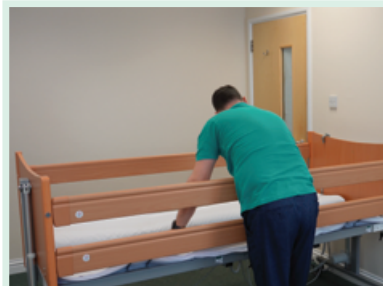

2. Place the protector on the bed ensuring the mesh panel is at the bottom. The hook and loop fastenings of the foam panels should be facing up.

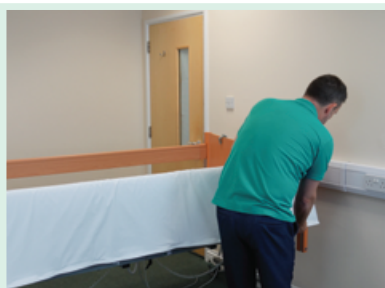

3. Fold edge (not attached to mesh panel) over the top rail and fasten to the mesh edge under the bottom rail using the hook & loop fastenings. Repeat on the other side.



4. Disconnect 3 of the central mesh panels by gently pulling apart the hook & loop fastenings on the calf, thigh & back rest panels, leaving the seat section (2nd from top) connected.



5. Profile the bed and drop the disconnected panels vertically between the bed frame and profiling sections.



6. Re-join the 3 split mesh panels underneath the bed using the hook and loop fastening, making sure they are not caught on the profiling sections.



7. Return the profiled bed back to a flat position and return the mattress.

When using any bed accessories, make sure you have done your own risk assessment or follow the guidance of someone who has done a risk assessment. Check all equipment for signs of wear and tear prior to use.

2022-v. 1



MIP

The Softer Side of Healthcare

OFFICE: MIP UK 2nd Floor Clipper House, Leighton Industrial Park, Billington Road, Leighton Buzzard, Beds, LU7 4AJ • Tel.:01525 218146 • mipuk.co.uk