



Mesh Bed Rail Protectors

Fitting Instructions



Scan the QR code to watch the fitting video.

Ensure the bed is at a safe and comfortable working height for yourself.



1. Lay the protector on the bed ensuring the zip on the edge close to you is facing down.



2. With the side rail at the raised position feed one edge through and under the lower rail.



3. Fold the other edge over the top rail and fasten the top edge to the top of the mesh panel using the zip closure.



4. Fold up the lower edge around the front of the lower rail, and fasten to the bottom of the mesh panel using the lower zip closures.

When using any bed accessories, make sure you have done your own risk assessment or follow the guidance of someone who has done a risk assessment. Check all equipment for signs of wear and tear prior to use.

2022-v. 1



MIP

The Softer Side of Healthcare