

Swift®

Swift® UltraSlide Standard Bottom Sheet

Product Code: PTD-LS/S/UK

Size: To fit Standard size beds

Fits Mattress size: Width: 32" to 36" (81 to 91 cm) Length: 78" to 84" (198 to 213 cm) Depth: 4" to 8" (10 to 20 cm)

Washable / Re-usable In-Situ Patient Repositioning System

When used with Swift® Slider (separate item)



 **MIP**

IPTDLSSUKv1

UK
CA

MD



MIP (UK) Ltd

2nd Floor, Clipper House, Leighton Industrial Estate,
Billington Road, Leighton Buzzard, Bedfordshire. LU7 4AJ
Tel: 01525 218146

mipuk@mip.ca - www.mipuk.co.uk



EC REP

MIP Europe GmbH

Liebigstraße 71, 22113, Hamburg, Deutschland
Tel: +49(0)40-73 93 20 – 0

info@mip-europe.com - www.mip-europe.com



REF

PTD-LS/S/UK

LOT



Pack Qty: 1



DO NOT USE TO LIFT PATIENT

Wash Swift prior to first use

Inspect for signs of wear and damage before use

Disposal – always follow local guidelines

Contact manufacturer in case of serious incident

Consult your moving & handling manager / guidelines before use



Swift® UltraSlide Bottom Sheet Instructions

The Swift® UltraSlide Bottom Sheet and Swift® Slider (separate item) is a washable / re-usable In Situ Patient Repositioning System suitable for both standard and bariatric uses. UltraSlide Bottom Sheet Sizing:

- Standard: Width: 32" to 36" (81 to 91 cm) Length: 78" to 84" (198 to 213 cm) Depth: 4" to 8" (10 to 20 cm)
 - Bariatric: Width: 42" to 50" (107 to 127 cm) Length: 78" to 84" (198 to 213 cm) Depth: 5" to 9" (13 to 23 cm)
- Both are suitable for use with the Standard Length and Full-Length Swift® Slider

Before using an in-situ repositioning system an appropriate risk assessment should be carried out by a competent person and recorded in the patient's care plan. The assessment will consider the user, the patient, the task and the environment in which the in-situ repositioning system is to be used. Safe patient handling practices should always be adopted by following your local hospital / facility patient handling guidelines available from your Patient Moving and Handling Advisor. If in doubt always consult your Patient Moving and Handling Advisor or Trainer before use.

- **Not for Lifting** – Using Swift® for repositioning should be accomplished without lifting all or most of the patient's weight.
- **Inspect before use** – Swift® should be inspected prior to use for signs of wear-and-tear and replaced, as necessary. Fabric, thread and stitch degradation caused by continuous use will take place over time (as normal wear and tear and/or washing) and will reduce the sliding effectiveness. Check for rips and tears and then rub a Swift® Slider on an UltraSlide® bottom sheet and check how well it slides.
- **Slip Hazard** – Swift® Slider and UltraSlide® Slide sheets should never be left on the floor as they are slippery and will create a slip hazard.
- **Tissue Viability Considerations** – Swift® in-situ patient repositioning system does not present a tissue viability hazard in normal use with healthy patients. Extra care should be taken when using this system with vulnerable patients who may be predisposed to tissue viability issues. You can mitigate these risks by adopting correct patient handling techniques. The use of this system will eliminate the need for insertion and removal of slide sheets, reducing risks and minimizing disturbance for the patient.

Instructions for Use

Place UltraSlide bottom sheet over mattress, as you would a normal fitted sheet ensuring it is the right way up (with the hem of the blue panel against the mattress) and stretch the sheet into the corners to prevent "hammocking".

- If you have an alternating air pressure reducing mattress, leave the corner where the inlet pipe is till last. The deep sides and extensive overlap on each corner of the sheet means fabric can be gently manipulated around the pipe
- When used without the Swift® Slider, the UltraSlide bottom sheet can be used with patients/residents who are more independent or who need simple instructions or facilitation techniques with bed mobility as the low-friction surface allows them to move themselves more easily in bed.
- If assistance is required, use the Ultra slide bottom sheet in conjunction with the Swift Slider (see instructions for Swift Slider for full instructions for use and basic manoeuvres).

Laundering & Cleaning

- Swift® can be laundered following standard hospital laundry guidelines HSG(95)18.
 - The use of bleaching agents should be avoided.
 - Maximum recommended washing temperature 95° C.
 - Tumble Dry on a medium to high heat

Disposal Instructions

- Please dispose of in a responsible manner in accordance with local linen waste guidelines, considering that the product may be contaminated after use and may need to be disposed of as clinical waste.